



## Usage Instructions – Airway Lubricant

Welcome to your new **7-SIGMA Simulation Systems (7S<sup>3</sup>) Torso Airway Trainer**. To achieve the best experience with our Torso Airway Trainer, we recommend the followings:

- A. To prepare the Torso Airway Trainer for a class, please take a few moments to lubricate it properly.
  - 1. Put 2 – 3 squirts of **7S<sup>3</sup> Lubricating Spray** down the tongue.
  - 2. Put the lubricant on your finger and wipe it over the posterior oral pharynx.
  - 3. Take an endotracheal tube and spray it with the airway lubricant, covering the balloon. Then pass the tube forward and backwards thru the vocal cords. This will lubricate the opening of the trachea.
  
- B. Follow these steps in class:
  - 1. Lubricate the nasal airway as you would for a patient.
  - 2. The oral airway may need to be lubricated on occasion.
  - 3. Lubricate an iLMA, Supreme LMA, Classic LMA, or an iGEL as you would if inserting into a patient.
  - 4. To insert a King Tube, inflate the balloons and use **7S<sup>3</sup> Lubricating Spray**, to lubricate both balloons.
  - 5. Use the same method to lubricate an endotracheal tube before insertion.

If you have any questions regarding the use of the **7S<sup>3</sup> Torso Airway Trainer/Airway**, please do not hesitate to contact us.

Technical Support  
Email: [technicalsupport@7-sigma.com](mailto:technicalsupport@7-sigma.com)  
Phone: 1.888.722.7224

Rev. 1.1  
5/22/2018